

STATSEAL[®] DISC

Today's Technology to Keep
Access Sites Dry and Intact




Application includes 1 StatSeal Disc.

StatSeal Disc, when combined with manual pressure, forms a seal over a wound or procedural site.

Instructions for Use

Product Application:

1. Prepare the skin around insertion site according to facility protocol.
2. Open sterile packaging and remove disc from center of the protective foam square using aseptic technique.
3. Place the disc around the catheter, with the brown side towards the patient's skin and the slit perpendicular to the catheter. The edges of the slit should approximate one another to the extent possible. The foam side should be facing up.
4. Hold continuous pressure on top of the foam for 2 minutes to form the seal; increase hold times as clinically necessary.
5. Secure the line per protocol.
6. Cover with a transparent dressing following facility protocol.
7. Remove disc with next dressing change.

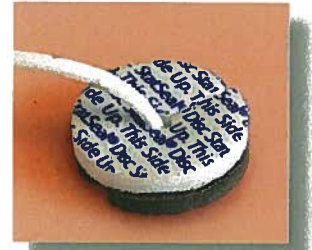

Suggested Protocol for Dressing Change:

1. Remove transparent dressing with non-sterile glove.
2. Apply sterile gloves and working from the insertion site down.
 - a) Remove the StatSeal disc.
 - b) Use sterile gauze to brush away any loose particles.
 - c) Remove the securement device.
 - d) Secure the catheter to the patient with tape or steri-strip to avoid catheter malposition.
4. Open the sterile field.
5. Apply a second set of sterile gloves.
6. Clean around the insertion site with CHG swab then allow site to fully dry.
7. After assessment of the insertion site:
 - a) Apply skin prep
 - b) Secure the catheter
 - c) Apply the transparent dressing and label the site per facility protocol

Suggested Protocol for Line Discontinuation:

1. Remove the transparent dressing and securement device from the site.
2. Remove the disc and brush away any remaining product.
3. Remove line per protocol.
4. Dress site per facility protocol. Additional StatSeal may be applied for bleeding as necessary.