

MRSA/BP CUFF CLEANING PROCEDURE

Supplies:

70% Isopropyl alcohol wipes or PDI wipes

Paper Towels

Apron

Surgical mask

Gloves

Procedure:

1. Double bag BP cuff in home
2. Bring to designated dirty utility room in office
3. Employee performing cleaning will don apron, gloves, surgical mask
4. Line countertop with paper towels
5. Remove BP cuff from bags
6. Place on top of paper towels
7. Wipe with 70% isopropyl alcohol wipe or PDI wipe and leave for 2 minutes
8. Flip BP cuff over
9. Wipe with 70% isopropyl alcohol wipe or PDI wipe and leave for 2 minutes
10. Wipe dry
11. If BP cuff is cloth allow to air dry completely prior to storing
12. Store BP cuffs in clean clear plastic bag and mark as clean with date
13. BP cuffs may be redistributed in clear plastic bag for use