

## CADD SOLIS Parameter Changes / Titrations

- Make sure pump is turned on & stopped
- Press the soft button under PROGRAM
- Highlight the parameter to be adjusted
  - Scroll up/down with the arrow buttons to highlight the parameter to be adjusted
  - Once highlighted, press SELECT
- Enter code with up/down arrows and the SELECT button for each numeral
  - Partner Pharmacies: 201
  - Chartwell: 061
  - Press the soft button under ACCEPT VALUE
- Use the up/down arrow buttons to adjust the parameter setting
  - Press the soft button under SAVE
- Make any additional parameter changes by highlighting the parameter and pressing the SELECT button selecting and adjusting with the up/down arrow buttons as above.
- Press the soft button under BACK (will take you to the home screen)
- Press the Stop/Start button
  - Press the soft button under REVIEW
    - Review each setting & accept each value
    - Press the soft button under ACCPE T VALUE for Each parameter
    - Must scroll through each parameter to accept each value and get a green Check Mark ✓ next to EVERY Parameter.
    - If you fail to get a green checkmark ✓ for each value, when the pump is shut down the setting will revert to the previous settings.
- Then press the soft button under NEXT
  - The pump will ask if you want to start the pump, press the soft button under NO
- Press he soft button under TASKS
- Press the soft button under LOCK KEYPAD
  - Lock icon will change to the locked position
- Press the soft button under BACK
- Press the soft button under PROGRAM
  - Review the pump settings for accuracy
  - Recommend a second check or read back to another clinician to verify the setting are correct.
  - Some agencies take a picture of the pump parameters for documentation
- You can now start the pump or power down the pump