

## 3.5 Administering Intramuscular (IM) and Subcutaneous (SQ) Injections

### Equipment:

1. Equashield Syringe Unit Prepared With Drug
2. Equashield Luer Lock Adaptor (LL-1)
3. Injection needle

### Procedure:

Follow aseptic technique and facility guidelines for safe handling and drug administration, confirming all connections are tightened securely. Follow facility protocol for disinfection of all access ports and Equashield components.

1

Luer LL-1  
to injection  
needle



2

Remove the cap  
from Syringe Unit



3

Disinfect LL-1  
membrane per  
facility protocol



4

Align red to red  
and engage  
Syringe Unit to  
LL-1 (Push On)



5

After injection is  
complete dispose of  
per facility protocol



NOTE: Facility protocol may vary. Please follow local safe handling guidelines and your facility protocols for aseptic handling.