

EQUASHIELD®


IV Push/Bolus Administration GUIDE

For Safe Nursing Administration

EQUASHIELD COMPONENTS
REQUIRED TO ENSURE SAFE
ADMINISTRATION:
LL-1 ADAPTOR



1

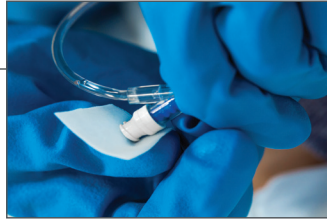

Disinfect the access port per facility protocol

2



Luer Lock LL-1 Adaptor to the access port

3



Pull **Protective Plug (PP-1)** from the Female LL Connector (FC-1)

4



Align **red to red**. Slide the FC-1 onto the LL-1 and administer

5



To safely disconnect:
Depress lever on LL-1. Pull FC-1 from LL-1



Note: Facility policy may vary

EQUASHIELD®

99 Seaview Blvd.
Port Washington,
NY 11050, USA

Toll Free: 855-378-2744
Clinical Support Line: 516-684-8200
Fax: 516-684-8202

clinical@equashield.com
www.equashield.com

EQL-1601.1